Wolfram Droh GmbH

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REV GA 2024-01

Yuyao Shengchang Medical Instrument Factory
Simen-Town
315400 Yuyao City, Zhejiang
China

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GB Contents

1	General	Page	20
2	Safety and Responsibilities	Page	20
3	Modification of the product	Page	20
4	Before using the product	Page	21
5	Using the product	Page	22
6	Product maintenance/Reprocessing	Page	22
6.1	Warnings	Page	22
6.2	Restrictions due to reprocessing	Page	23
6.3	Preparation at the place of use	Page	23
6.4	Preparation for cleaning	Page	23
6.5	Manual cleaning and disinfection	Page	23
6.6	Automatic cleaning and disinfection	Page	24
6.7	Drying	Page	24
6.8	Maintenance, control and inspection	Page	24
6.9	Packaging	Page	25
6.10	Sterilisation	Page	25
6.11	Additional Information	Page	25
6.12	Hygienic cleaning under homecare conditions	Page	25
7	Accessories and replacement parts	Page	26
8	Repairs	Page	26
9	Disposal	Page	26
10	Explanation of Symbols	Page	43

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1 General

- The product may be used by trained personnel as well as by lay persons.
- The product is intended to be used to supply medical oxygen within homecare oxygen therapy as well as in medical settings.
- The following instructions must be followed by untrained users.
- These instructions reflect the state of the art at the time of publication.
- This product complies with the provisions of the EU Directive 93/42/EEC (Medical Device Directive) and Medical Device Regulation (EU) 2017/745.
- The instructions for use comply with DIN EN 82079-1.

2 Safety and Responsibilities

IMPORTANT! READ CAREFULLY BEFORE USE
KEEP FOR FUTURE REFERENCE

- Please observe the following instructions and care instructions before use.
- For traceability purposes, the product may only be stored in its original packaging. If this is not possible for the user, the user is responsible for correct labelling and traceability.
- If in doubt about the use and connection of the product, please contact customer service.
- If additional oxygen is administered care should be taken to ensure that smoking and other naked flames are forbidden in the vicinity of oxygen-carrying equipment and fittings.
- The product draws air from the immediate surrounding and must therefore not be used in a toxic atmosphere.
- Keep the product free of oil and grease, as this could present an explosion hazard. This includes the exposure to cleaning alcohols, hand cream and adhesives – in contact with highly compressed oxygen explosive reactions may occur.
- Repeated use can lead to hardening and cracking. In this case the product must be disposed of according to chapter 9.
- When used in patients with a suspected prion disease, there may be a high risk of infection.

It lies within the discretion of the physician to properly dispose of the product according to chapter 9 or to reprocess it according to national regulations (see chapter 6).

3 Modification of the product

- Apart from the described maintenance instructions, the product must not be modified or repaired by the operator!
- The manufacturer's responsibility is limited to the dimensional and physiological suitability of the product.

- Improper use, modifications to the product or negligence will void any warranty. This also applies to unauthorised repairs. Normal signs of wear and tear on the device are also excluded.

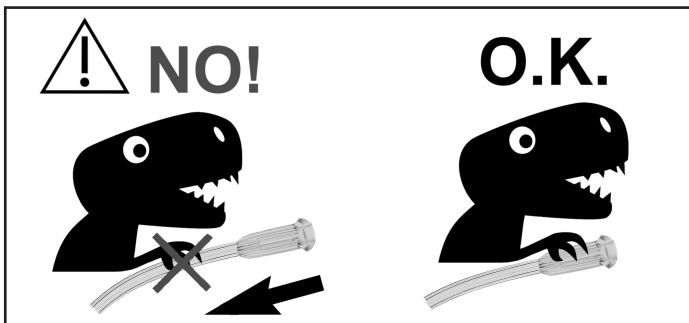
4 Before using the product

- Do not expose the product to direct sunlight or other light sources. Store and transport the product in a dry and dust-free place in its original packaging at -10°C to $+50^{\circ}\text{C}$.
- Remove all delivered parts from the packaging. If necessary, remove the respective plastic covers and dispose of the packaging according to the current national regulations.
- Verify that parts or components do not show any signs of hardening, damage, breakage or cracks. If any of the conditions described above are present, the device must not be put into operation and must be disposed of in accordance with chapter 9.
- In case of doubt perform a comparison check with a brand-new product.

Damaged products must not be used!

After consulting customer service, the product must be disposed of according to chapter 9.

- The reusable products are delivered non-sterile and must therefore be unpacked and fully reprocessed (according to chapter 6) before use.
- The processing instructions under chapter 6 must be followed to avoid any exposure to danger! In case of questions please contact customer service.
- To test for leakage, the tubing can be briefly submerged under water with the oxygen supply attached. If air bubbles appear on the product, do not use.
- The product contains silicone.
- To attach and detach the tubing, the tubing must be held at the connector to prevent cracks from forming between the connector and the tubing.

- **Assembly of the oxygen safety tubing:** Connect the fittings to the corresponding connectors and/or directly to the oxygen source.
- **Connecting the nasal cannula:** Connect the fittings to the appropriate connectors or directly to the oxygen source.
Insert the nasal prongs at the other end of the tubing directly into the nostrils. Place the ends of the silicone tubing over the ears and secure these under the chin using the bolo. This should be pulled up to the chin for security.
- For total lengths over 15 m, a water trap should be used to collect any condensation.
- When completely assembled, check that all parts are connected tightly.
- The product is now ready for use.

5 Using the product

- The product must not come into contact with sharp objects or rub against an edge during use.
- After successful assembly, the product can be attached to the oxygen source. Please note the requirements of any other connection devices and products.
- The flow rate can be set to 1–10 lpm.

6 Product maintenance/Reprocessing

- The product is reusable subject to risk assessment by the user.
- The service life of reusable products is generally determined by wear and tear and damage caused by use of the product.
- However, the service life is limited to a maximum of two months or 8 reprocessing cycles from the first usage onwards.
The actual wear and tear and the associated number of reprocessing cycles will vary.
For this reason, a functional test must be carried out before each use to determine possible material fatigue or impaired functionality.
- Under homecare conditions and manual disinfection the durability of the product can be greatly reduced.
- Observe the following procedures for reprocessing (cleaning and disinfecting). These are validated by the manufacturer.
- A complete reprocessing cycle always consists of cleaning and disinfection (manual or mechanical) and sterilization.

6.1 Warnings

- The product is delivered non-sterile.
- The product must be processed (cleaned and sterilised) before initial use and before each repeat use. At the latest once a week for constant use.
The delivered packaging cannot be sterilized!
- Do not use any pointed or sharp objects as well as hard brushes, scouring agents or other materials that could damage the surface materials!

- The manufacturer accepts no responsibility for damage due to incorrect processing.
- The processing of the product lies within the clinical responsibility of the user.
- No residues may remain in or on the product.
- Do not use damaged products. If necessary, contact customer service.

6.2 Restrictions applicable to reprocessing

- The use of oils and greases is not recommended for silicone products.
- Do not use disinfectants based on formaldehyde, glutaraldehyde, glucoprotamine, cocosdipropylenetriamine, quaternary ammonium compounds, phenol derivatives or iodine-cleaving compounds.
- In addition, it must be ensured that no hard or sharp-edged objects come into contact with the individual parts during disinfection and that the parts are rinsed thoroughly afterwards.

6.3 Preparations at the place of use

- It is recommended to rinse the product thoroughly with warm water and a diluted sodium bicarbonate solution (8–10% by volume) immediately after use until all visible foreign matter has been removed.
- Do not let any residues become dried-on.
- Immediately after use, manual disinfection should be performed to reduce the risk of infection for users.

6.4 Cleaning preparation

- Loosen the bolo and remove the nasal prongs from the nostrils.
- Disconnect the connectors of the silicone tubing from the connectors and/or the oxygen source.

6.5 Manual cleaning and disinfection

- Solely manual reprocessing is not recommended due to the lower effectiveness and reproducibility. It should only be used if no automated procedure is available.
- Use fully demineralised water.
- Cleaning and disinfecting solution must be prepared before each manual cleaning cycle. Please follow the manufacturer's instructions for the solution.
- A solution of Sekusept Aktiv is recommended.
- Place the product in the cleaning and disinfecting solution. Pay attention to the manufacturer's recommended concentration and soaking time.
- Even when cleaning manually, make sure that all cavities are properly rinsed through.
- A small brush with soft bristles or a soft sponge can be used to help clean the product.
- After cleaning, rinse sufficiently with fully demineralised water to remove any residues.
- Avoid any water accumulation.

- Dry the product completely.
- Check visually for residues.
Repeat the rinsing process if necessary.

6.6 Machine cleaning and disinfection

- Use fully demineralised water.
- If using alkaline cleaning agents, neutralisation must be carried out first.
- Do not use any desiccants.
- It is recommended to use an automated/ mechanical cleaning and disinfection procedure.
- The product is suitable for machine processing/thermal disinfection ($\leq 93^{\circ}\text{C}$).
- It must be ensured that the selected washer-disinfector complies with ISO 15883.
- Follow the operating instructions of the respective device model.
- Use cleaning agents that are suitable for stainless steel and polysulfone.
- **Recommended cleaning agents:**
Cleaner: Sekumatic ProClean // Neutraliser: ECOLAB Sekumatic FNZ
- The operating cycle (cleaning, disinfection, rinsing, drying if necessary) must achieve the specified performance.
- Do not overload the basket and avoid shadowing.
- Load products with openings and recesses with open side facing downwards to avoid accumulation of cleaning water.

Instructions:

1. Place the products in the washer-disinfector / arrange around the spiral rack (ensure that all cavities are effectively flushed through).

Observe the loading pattern of your washer-disinfector if necessary.

2. Start a cycle with the following parameters: (e.g. VARIO TD)

- Pre-rinse with purified water 20°C // 1–4 min.
- Cleaning with alkaline cleaner max. 55°C // 5–10 min.
- Neutralisation 20°C // 2–6 min.
- Intermediate rinsing 20°C // 2–3 min
- Disinfection 93°C // 5 min.
- Drying max. 100°C // 30 min

3. Inspect for any residual soil. Repeat the cycle if necessary.

4. Inspect the products and prepare for sterilization.

6.7 Drying

- After cleaning, the products must be dried thoroughly (preferably supported by compressed air).
- Do not use drying agents or cloths which could leave residues on the silicone.

6.8 Maintenance, control and inspection

- Inspect the products after disinfection and before each use for intactness, leakage and cleanliness.

- Check the product for cracks and material changes.
- If residual contamination is still visible, repeat the entire procedure until no more residues are visible.

6.9 Packaging

- After processing, package the product according to ISO 7153.
- Store sterilised products in sterile packaging in a closed cabinet, protected from dust, light, moisture and temperature fluctuations.

6.10 Sterilisation

- The product must be sterilised before each use.
- Before sterilisation, pack the products in a suitable container or in suitable sterilisation wrapping.
- The correct size of container or wrapping must be chosen.
- Use a sterilisation indicator for the wrapping and note the sterilisation and expiry date on the wrapping.
- The product is only suitable for steam sterilization in a fractionated vacuum process.

	Quick programme	Normal programme
Maximum exposure time:	5 mins	20 mins
Maximum temperature:	134 °C	121 °C
Maximum pressure:	3 bar	2 bar

- Follow the instructions of the autoclave manufacturer carefully.
- After steam sterilisation, the product must be allowed to cool down at room temperature.
- The manufacturer accepts no liability for risks arising from improper sterilisation (contamination).

6.11 Additional Information

- Frequent sterilisation can accelerate the natural ageing of the product. In addition, the service life of sterilised products depends very much on carefully performed pre-treatment (cleaning and disinfection).
- When sterilising several products in one sterilisation cycle, the maximum load of the steriliser must not be exceeded.

6.12 Hygienic cleaning under homecare conditions

- Place the tubing in warm tap water with a little washing up liquid for about 5 minutes.
- Remove any dried-on residues or other dirt with a soft, damp dishcloth. Consider using mild soap.
- Rinse all parts thoroughly with clean water.
- Disinfect the tubing at least once a week.

- To do this place the product in boiling (ideally distilled) water for about 20 minutes. Make sure that it is completely covered with water.
- Then dry the tubing with a lint-free cloth.
- Then use your oxygen source to allow oxygen bursts to flow through the tubing at short intervals until the moisture in the tubing is removed.
- Then let the remaining moisture dry out at room temperature.
- Before use, check that there is no more condensation in the tubing.

7 Accessories and spare parts

- In case of compatibility questions please contact customer service.

8 Replacement parts and repairs

- Repairs must only be carried out by the manufacturer or by authorised persons.
- In case of functional or material problems, contact customer service immediately.
- If the cause of the error cannot be eliminated, the product must not be put into operation. After consulting the customer service, the product must be disposed of according to chapter 9 or returned to the sales distributor.
- In the event of a return for repair or replacement the entire reprocessing process must be carried out beforehand in order to exclude any risk to the manufacturer's employees.

We reserve the right to reject soiled or contaminated products for safety reasons.

9 Disposal

- If any of the above mentioned errors have occurred, the product must be disposed of.
- The product must be disposed of at the latest two months after first use or following 8 reprocessing cycles.
- Safe disposal is carried out in accordance with current national regulations.

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